Harmony™ Harmony™ Pump & Feed Set

No. 1 choice
of hospitals
& mothers

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Thank you for choosing the Harmony breast pump. Breast milk is the natural choice for your baby, but there are reasons and situations where breastfeeding is not possible. For over 50 years, it has been Medela's conviction that these mothers and babies can be supported. In close cooperation with breastfeeding experts, Medela, one of the market leaders, has researched and developed breast pumps that perfectly meet the needs of breastfeeding mothers. For mothers who insist on the best for their babies and themselves. Because you care.

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Table of Contents

4

1. Intended use/Population – Contraindication	5
2. Meaning of symbols	6
3. Important safety information	8
4. Product description	10
4.1 Harmony – light, convenient, clever	10
4.2 Product parts	11
5. Cleaning	12
5.1 Before using for the first time and after every use	12
5.2 Before using for the first time and once a day	14
5.3 Cleaning – summary	16
6. Preparing for pumping	18
6.1 Assembling the handle	18
6.2 Assembling the pump set	18
6.3 Placing the breast shield	20
7. Expression	22
8. Storing and thawing breast milk	24
8.1 Storage	24
8.2 Freezing	24
8.3 Thawing	24
9. Feeding breast milk	25
10. Troubleshooting	26
11. Disposal	27
12. Complete care during breastfeeding time	28
13. Technical specifications	143

1. Intended use/Population – Contraindication

5

The intended use and indications of breast pumps

Breast pumps are intended to be used by lactating women in the hospital or home environment for expressing breast milk. Breast pumps are indicated to alleviate the symptoms of engorgement by expressing milk. In case of mastitis, breast pumps support the healing process by removing breast milk from the affected breast. Breast pumps are indicated to relieve sore and cracked nipples and to bring out flat or inverted nipples. Also breast pumps enable mothers to provide breast milk for babies who cannot feed directly at the breast for a variety of reasons such as latch-on problems, cleft palate or premature babies.

Intended patient population/user

Adolescent paediatric or adult lactating women. Many women find it convenient, or even necessary, to use a breast pump to express and store their breast milk once they have returned to work, are travelling or are otherwise separated from their baby. Women are often delighted to learn that a breast pump can be used as a supplement to breastfeeding and that some pumps are designed to mimic the sucking of a nursing baby.

Contraindications

There are no known contraindications for the Harmony breast pump.

2. Meaning of symbols

6

The warning symbol identifies all instructions that are important for safety. Failure to follow these instructions can lead to injury or damage to the Harmony breast pump! There are several warning symbols. Each symbol is used together with a specific word. The warning symbol/word combinations stand for:

Caution

Failure to follow instructions of this type can lead to (minor) injury or contamination/growth of bacteria.

Note

Failure to follow instructions of this type can lead to material damage.

Info

Useful or important information that is not related to safety.

Symbols on the packaging

This symbol indicates that the material is part of a recovery/recycling process.

This symbol indicates a carton package.

This symbol indicates to keep the device away from sunlight.

This symbol indicates to handle the fragile device with care.

This symbol indicates the temperature limitation for operation, transport and storage.

This symbol indicates the humidity limitation for operation, transport and storage.

This symbol indicates the atmospheric pressure limitation for operation, transport and storage.

This symbol indicates to keep the device dry.

This symbol indicates that the packaging contains products intended to come in contact with food according to regulation 1935/2004.

This symbol indicates the unique GS1 Global Trade Item Numbers (GTIN).

This symbol indicates to follow the instructions for use.

This symbol indicates the manufacturer.

This CE symbol indicates the compliance of the Harmony breast pump 0123 with the essential requirements of the Council Directive 93/42/EEC of 14 June 1993 concerning medical devices.

Symbols on the device

This symbol indicates the recycling code (here 05 for material made of polypropylene PP).

This symbol indicates the recycling code (here 7 for all material that is not specified by the other recycling codes).

This code indicates the use of polypropylene.

This CE symbol indicates the compliance of the Harmony breast pump 0123 with the essential requirements of the Council Directive 93/42/EEC of 14 June 1993 concerning medical devices.

3. Important safety information

8

For your child's safety and health

CAUTION!

Failure to follow the safety information
on these pages can lead to danger.

Device

The Harmony breast pump is not heat-resistant: keep away from radiators and open flames.

Do not expose the Harmony breast pump to direct sunlight.

Check the breast pump's components for wear or damage.
Never use a defective device. Throw a part away at first signs of damage or weakness.

Important:

- I Plastic bottles and component parts become brittle when frozen and may break when dropped.
- I Also, bottles and component parts may become damaged if mishandled, e.g. dropped, over-tightened or knocked over.
- I Take appropriate care in handling bottles and components.
- I Do not use the breast milk if bottles or components become damaged.

Use

Use the Harmony breast pump only for its intended use as described in this manual.

Supervision is necessary when you use the Harmony breast pump in the vicinity of children. Keep all parts that are not in use out of children's reach.

Contact your lactation consultant or physician if you experience problems or pain.

This is a single user product. Use by more than one person may present a health risk.

4. Product description

10

4.1. Harmony – light, convenient, clever

By buying the Harmony breast pump, you have chosen well: Harmony is the first manual breast pump based on 2-Phase Expression technology. This technology mimics your baby's natural sucking rhythm. The pump's ergonomic handle allows you to alternate between short movements to stimulate and longer ones, to express the milk. Just like your baby sucks when it is being fed on your breast. As a result, you can express more milk in less time.

The Harmony breast pump is ideal for occasional usage. For example, if you need to leave your baby alone for short periods, e.g. to do sports or have an evening out. Or if you want to involve your partner in breastfeeding. You can also use Harmony in addition to a rental pump. Harmony can also help to overcome health problems, such as sore, flat or inverted nipples.

The main features of your Harmony breast pump are:

- I Easy to assemble, use and clean
- I Silent and discreet pumping
- I Light to transport, leaving you independent and mobile

Harmony includes the breast pump only.

Harmony Pump & Feed Set includes the breast pump, Calma Solitaire and four disposable nursing pads.

Throughout the instructions for use the name Harmony refers to both versions of the pump unless otherwise specified.

4.2 Product parts

You can find spare parts and accessories available for order in chapter 12.

* for other breast shield sizes, see chapter 12

5. Cleaning

Caution

| Take apart and wash all parts that come in contact with the breast and breast milk immediately after use, to avoid the drying up of milk residues and to prevent growth of bacteria.

| Only use drinking-quality water for cleaning. To avoid contamination, do not touch the inside of the parts after cleaning.

| Do not clean the pump set with a sharp object (such as a toothpick).

| Take care not to damage parts of the pump set during cleaning.

| Some parts of the Harmony pump set are very small (e.g. the O-ring or the valve membrane). Check that no parts are missing when you reassemble the pump set after cleaning.

Note

| The pump set requires no maintenance.

| If you clean the individual parts of the pump set in a dishwasher, parts may change in colour. This has no impact on the function.

| Do not leave (parts of) the pump set in direct sunlight or heat.

| Do not use chemical sterilisers.

5.1 Before using for the first time and after every use

1

Disassemble the pump set into its individual parts.

4

Rinse all parts with cold, clear water (approx. 20 °C) for 10–15 seconds.

2
Rinse all parts with cold, clear water (approx. 20 °C).
Check the parts for damage and throw away at first signs of damage or weakness.


3
Clean all parts with plenty of warm soapy water (approx. 30 °C).

5
Dry with a clean cloth or leave to dry on a clean cloth.

or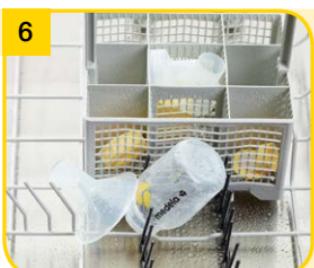

6
As an alternative to steps 2, 3 and 4.

If you clean the individual parts of the pump set in the dishwasher, place them on the top rack or in the cutlery section.

5. Cleaning

Caution

- I Do not drop the parts of the pump set into boiling water.
- I To avoid contamination, do not touch the inside of the parts after sterilisation.
- I Take apart and wash all parts that come in contact with the breast and breast milk immediately after use to avoid drying up of milk residues and to prevent growth of bacteria.**
- I Only use drinking-quality water for cleaning.

Note

- I When you boil the parts of the pump set, you could add a tea-spoon of citric acid to avoid a build-up of limescale.
- When you use citric acid, minor traces might remain on the pump parts. Use drinking-quality water to wash them off. Dry the parts as shown in figure 3 to the right.

5.2 Before using for the first time and once a day

1

Disassemble the pump set into its individual parts.

3

Dry with a clean cloth or leave to dry on a clean cloth. Store the clean parts in an air-permeable, clean bag/container, or in an air-permeable, clean paper/cloth until next use.

2a**or****2b**

Cover all parts with water and boil for five minutes.

Use a Quick Clean* microwave bag in accordance with the instructions.

5. Cleaning

16

5.3 Cleaning – summary

Before using for the first time and after every use

Rinse with cold, clear water (approx. 20 °C), then clean with plenty of warm soapy water (approx. 30 °C) and finally rinse with cold, clear water for 10–15 seconds;

✓ ✓ ✓ ✓ ✓ ✓ ✓ ✓ ✓

Or wash in the dishwasher (on the top rack or in the cutlery section).

✓ ✓ ✓ ✓ ✓ ✓ ✓ ✓ ✓

Before using for the first time and once a day

Cover with water and boil for five minutes;

✓ ✓ ✓ ✓ ✓ ✓ ✓ ✓ ✓

Or use a Quick Clean microwave bag in accordance with the instructions.

✓ -- ✓ ✓ ✓ ✓ ✓ ✓

6. Preparing for pumping

Caution

I Wash your hands thoroughly (for approx. one minute) with soap and water before you touch your breast and the pump parts.

Note

- I Only use Medela original accessories.
- I Check that the (white) O-ring is still placed around the stem:

The O-ring makes a sealed connection between the handle and the 2-component connector possible. This is necessary to build up a vacuum.

Info

- I Carry out all steps with care and assemble the pump set correctly for optimum performance.

6.1 Assembling the handle

1


Insert the stem (with the O-ring) into the hole on the upper side of the diaphragm. The stem and diaphragm are parts of the pump's handle.

6.2 Assembling the pump set

1


Carefully insert the breast shield into the top opening of the connector.

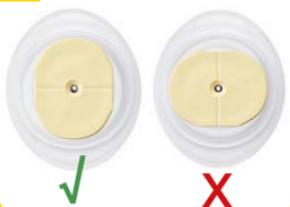2

Make sure the oval shaped base of the stem has the right position in the diaphragm.

3

Place the diaphragm into the pump handle. Press down firmly around the entire edge to ensure a proper seal.

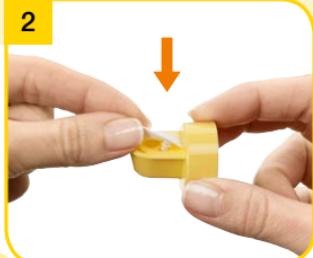2

Place the valve membrane in a horizontal position on the valve head. Carefully push the knob on the membrane's back into the smallest hole of the head until it engages. The membrane must lie flat on the head.

3

Push the valve head onto the inner ring of the connector. The valve head must be positioned sideways (as you can see in the above picture).

6. Preparing for pumping

20

6.2 continued

4

Screw the bottle onto the connector.

5

Insert the stem of the handle assembly into the opening on top of the connector. Push until the stem locks into place.

6.3 Placing the breast shield

1


Wipe your breast with a warm cloth (do not use alcohol). Place the breast shield on your breast so that your nipple is properly centred in the tunnel.

2

Hold the breast shield onto your breast with your thumb and index finger. Support your breast with the palm of your hand.

Does your breast shield fit?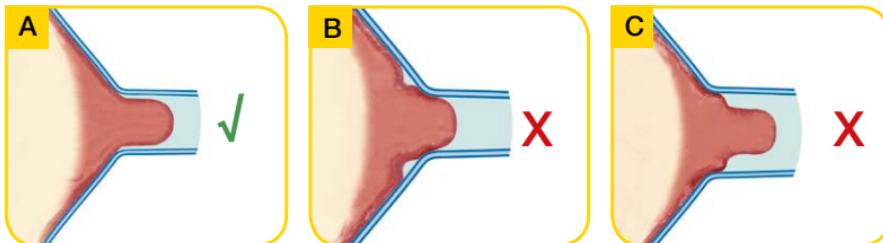

It is important that the breast shield fits correctly. A properly fitting breast shield ensures comfort and efficient milk expression. Figure A above shows a breast shield that is the correct size. The breast shield in figure B is too small, the one in figure C too large.

Answer the following questions to find out whether your breast shield fits correctly:

- █ Is your nipple moving freely in the tunnel of the shield?
- █ Is no or a minimal part of your areola being pulled into the tunnel of the breast shield?
- █ Do you see a gentle, rhythmic motion in the breast with each cycle of the pump?
- █ After pumping, does your breast feel much less full, with no areas of hardness?
- █ Is your nipple pain-free?

If the answer to all of these questions is "Yes", then your breast shield fits correctly. If the answer to any of these questions is "No", or if you experience problems or pain during expression, your breast shield may not be the right size. Medela offers five different sizes of the PersonalFit breast shield. Consult your breastfeeding and lactation consultant or midwife for advice.

7. Expression

22

Caution

- | Wash your hands thoroughly (for approx. one minute) with soap and water before you touch your breast and the pump parts.
- | Contact your midwife or breast-feeding and lactation consultant if you can express only minimal or no milk or if expression is painful.

Note

- | Do not hold the pump set by the bottle. This can lead to engorgement and blockage of the milk ducts.
- | Only fill the bottle to the 150 ml mark.

1

Start pumping with stimulation: press down the smaller part of the handle. This is the part with the milk drops on top. Press and release quickly, again and again, until your milk starts to flow.

→ The Stimulation mode mimics your baby's initial rapid sucking. It is designed to stimulate the milk flow.

2

Once your milk starts to flow, switch to expression: press down the lower, longer part of the handle.

By doing so, you build up a vacuum that facilitates the milk flow. You control the level of the vacuum with the speed of the handle stroke and the compression that is created. Decide yourself which pumping rhythm is the best for you.

→ The Expression mode mimics your baby's slower, more relaxed sucking. It is designed to maximise the milk flow.

3

Stop pumping when you feel that the milk does not flow anymore or when the bottle is full. Use the stand to prevent the bottle from tipping over.

4

Clean as described in chapter 5

Unscrew the bottle from the connector. Close the bottle with the multi lid. Store the breast milk (follow the instructions in the next chapter, chapter 8).

8. Storing and thawing breast milk

8.1 Storage

Storage guidelines for freshly expressed breast milk (For healthy term babies)

Room temperature	Refrigerator	Freezer	Thawed breast milk
4–6 hours at 19 to 26°C (66 to 78°F)	3–8 days at 4°C (39°F) or lower	6–12 months –18 to –20°C (0 to –4°F)	In refrigerator for 10 h max. Do not refreeze!

I Do not store breast milk in the door of the refrigerator. Instead choose the coldest part of the refrigerator (this is at the back of the glass shelf above the vegetable compartment).

8.2 Freezing

I You can freeze expressed breast milk in milk bottles or “Pump & Save”* bags.
Do not fill the bottles or bags more than $\frac{3}{4}$ full to allow space for possible expansion.
I Label the bottles or “Pump & Save” bags with the date of expression.

8.3 Thawing

Caution

Do not thaw frozen breast milk or do not warm breast milk in a microwave or a pan of boiling water to avoid loss of vitamins, minerals and other important ingredients and to prevent burns.

I In order to preserve the components of the breast milk, thaw the milk in the refrigerator overnight. Alternatively, you can hold the bottle or “Pump & Save” bag under warm water (max. 37 °C).
I Gently swirl the bottle or “Pump & Save” bag to mix any fat that has separated. Avoid shaking or stirring the milk.

* More information at www.medela.com

9. Feeding breast milk

25

Info

Medela recommends Calma for feeding breast milk.

With Calma

- I your baby can feed, pause and breathe regularly.
- I milk will flow when a vacuum is created.
- I your baby's natural feeding behaviour will be maintained, supporting easy transition back to the breast.

Calma – unique feeding solution for breast milk.

You can find detailed information about Calma on the Medela website (www.medela.com).

As a result of its comprehensive research work, Medela is one of the first companies in the world to incorporate babies' breastfeeding behaviour into its 2-Phase Expression and Calma innovations.

2-Phase Expression mimics the baby's natural sucking rhythm to express more milk in less time. Short, stimulating patterns are followed by longer ones, providing optimal milk flow.

The unique Calma breast milk feeding device allows the baby to pause and breathe during feeding whilst maintaining the suckling behaviour learned at the breast. Developed for a great breastfeeding experience.

* Calma is included in the Harmony Pump & Feed Set only. It is also available separately; please refer to chapter 12.

10. Troubleshooting

26

Problem	Solution
If there is low or no suction	<ul style="list-style-type: none"> Make sure the yellow valve head and the white membrane are clean and not damaged. Make sure the membrane lies flat on the valve head. Make sure the valve head is positioned sideways on the connector. Make sure the components of the handle are assembled correctly. Check all connections. Make sure the breast shield forms a complete seal around the breast.
If there is no milk flow	<ul style="list-style-type: none"> Ensure your breast pump is assembled correctly and there is suction. Relax and take a 10-15 minute break if expression is not achieved after two consecutive pumping sessions. Consult your breastfeeding and lactation consultant or midwife if expression still does not occur.

In the case of a product failure, please contact your local subsidiary.
You can find the addresses on the reverse of this instruction manual.

11. Disposal

The Harmony breast pump is made of plastics.
You must dispose of it in accordance with local regulations.

12. Complete care during breastfeeding time

28

The following product variants, spare parts, consumables and accessories for the Harmony breast pump are available from Medela distribution partners. You can order a product by specifying the product name or the corresponding product (article) number.

Product variants

Article number	Product
005.2058	Harmony EN/KO/TW
005.2071	Harmony Pump & Feed Set EN/KO/TW

Spare parts

Article number	Product
099.0259	Harmony Handle
099.0260	Harmony Membrane complete

Consumables

Article number	Product
800.0727	Valves & Membranes - Extra parts for use with Harmony, Swing, Base, Mini Electric, Lactina and Symphony. Consists of two valve heads and six membranes.

Accessories

Article number	Product
008.0362	PersonalFit breast shield S (21 mm)
008.0363	PersonalFit breast shield M (24 mm)
008.0364	PersonalFit breast shield L (27 mm)
008.0365	PersonalFit breast shield XL (30 mm)
008.0366	PersonalFit breast shield XXL (36 mm)
008.0072	Breast milk bottles 150 ml – 3 pcs.
008.0091	Breast milk bottles 250 ml – 2 pcs.
008.0318	Pump & Save breast milk bags
008.0148	Calma Solitaire

Other Medela products are available at www.medela.com

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Medela AG