## Instructions for use

### Pump & Save™ Breast milk bags

#### Bruksanvisning

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#### Brugsanvisning

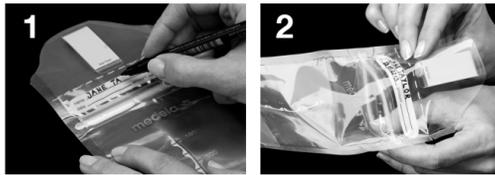
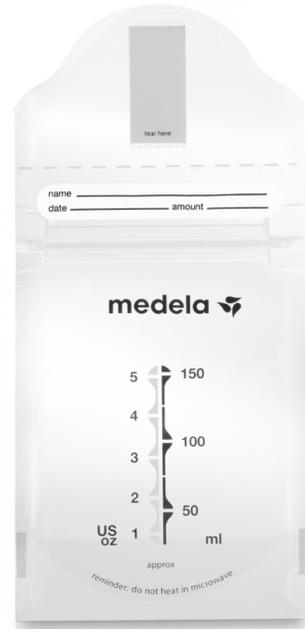
### Pump & Save™ Modermælksposer

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## EN

### Pump & Save™ Breast milk bags

Ideal to collect, store and freeze breast milk

#### Collecting breast milk

1. The label area allows you to write information on the bag.
2. Gently open the zip-top closure, taking care not to tear the perforated top.
3. Place the bag around the collar of the breast shield.
4. Release the yellow strap by peeling where indicated, and unfold.
5. Wrap the strap over the top of the breast shield.
6. Peel the paper tab off the end of the strap and stick to the opposite side of the bag. Caution: Keep the paper tab away from children (danger of swallowing).
7. Begin pumping. Do not fill the bag past the 150 ml mark.
8. When you have finished pumping, remove the bag by tearing off the perforation just above the label area. The pleated bottom of the bag will allow it to stand up safely by itself while you put away your pump.
9. To remove the strap from the breast shield, tear the strap and discard the top portion of the bag.
10. Flatten the bag above the milk to remove excess air, then zip to close.

#### Hygiene

Wash hands thoroughly with soap and water before touching the breast and pump parts. Avoid touching the inside of the breast milk bag. Clean the breast and surrounding area with a damp cloth. Do not use soap or alcohol which can dry the skin and cause cracked nipples.

#### Storing and thawing breast milk

Storage guidelines for freshly expressen breast milk (for healthy term babies)	
Room Temperature	4-6 hours at 19 to 26°C (66 to 78°F)
Refrigerator	3-8 days at 4°C (39°F) or lower
Freezer	6-12 months at -18 to -20°C (0 to 4°F)
Thawed breast milk	In refrigerator for 10 h max. Do not refreeze!

- Do not store breast milk in the door of the refrigerator. Instead choose the coldest part of the refrigerator (this is at the back of the glass shelf above the vegetable compartment).
- You can freeze expressed breast milk in milk bottles or „Pump & Save“ bags. Do not fill the bottles or bags more than ¾ full to allow space for possible expansion.
- Label the bottles or „Pump & Save“ bags with the date of expression.

#### Caution

Do not thaw frozen breast milk or do not warm breast milk in a microwave or a pan of boiling water to avoid loss of vitamins, minerals and other important components and to prevent burns.

- In order to preserve the components of the breast milk, thaw the milk in the refrigerator overnight. Alternatively, you can hold the bottle or „Pump & Save“ bag under warm water (max. 37°C).
- Gently swirl the bottle or „Pump & Save“ bag to mix any fat that has separated. Avoid shaking or stirring the milk.

Material:  
Outer Layer: Polyester  
Inner Layer: Polyethylen

Does not contain Bisphenol-A

#### Caution:

This is a single use product not intended to be reprocessed. Reprocessing could cause loss of mechanical, chemical and/or biological characteristics. Reuse could cause cross contamination.

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