Hydrogel Pads

Instructions for use
Gebruiksaanweisung
Mode d'emploi
Istruzioni d'uso
Gebruiksaanwijzing
Instrucciones de uso
Инструкция по применению

Οδηγίες χρήσης
הוראות שימוש
Bruksanvisning
Brugsanvisning
Käyttöohjeet
Bruksanvisning
Instrukcja użycia

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EN
Instructions for use

Wash hands before using the product.
 • Open pouch.
 • Remove gel pad from the plastic liner.
 • Gently place the gel side of the pad directly over your nipple.
 • Do not use if the pouch is open or damaged.
 • Prior to breastfeeding, remove the gel pad.
 • Place on a clean surface.
 • Gently wash the breast with warm water and pat dry.
 • Then start breastfeeding.
 • After breastfeeding, reapply the gel pad.
Do not use soaps or detergents to clean pad • Store gel pads in a cool, dry place • For more cooling sensation, gel pads can be stored in the fridge • Reusable for 24 hours • If gel pad becomes cloudy or milky, please discard and use a new one.
Optional: To improve soothing, briefly pass the gel side of the pad under a cool stream of water (1-2 seconds), shake off excess water and set aside for 2 minutes. Do not rinse more than twice a day.

Important Safeguards

Use only as indicated • Keep out of reach of children, especially the plastic liner • Extra care should be taken if using with cracked nipples or fungal infection • If nipples are cracked, bleeding or have a burning sensation, contact your lactation consultant or physician • To prevent nipple pain it is important that baby feeds at mother's breast in a correct position.
Ingredients: Water, glycerol, polymer

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© Medela AG 200. 1726007. 10/D