



No.1 choice  
of hospitals  
& mothers

# Disposable Breastmilk Bottles

# 80 ml

Ready-to-Use

### Collecting

- I To collect breastmilk with any Medela Pump Set
- I Compatible with any Medela Breastshield

### Breastmilk Management

- I To safely store expressed breastmilk in the refrigerator or freezer
- I Label bottle with name, date/time of pumping and amount of breastmilk
- I Suitable for pasteurisation

### Feeding

- I To directly feed expressed breastmilk

### Important Safeguards

- I The Disposable Breastmilk Bottle does not need cleaning prior to use as it is provided Ready-to-Use.
- I \* **Caution:** This is a single use product not intended to be reprocessed. Reprocessing could cause loss of mechanical, chemical and/or biological characteristics. Reuse could cause cross contamination.
- I Use a new Disposable Breastmilk Bottle for each pumping session.
- I Use Medela original accessories only.
- I Avoid touching the inside of bottles and lids.
- I Never leave accessories unattended and keep out of reach of children.
- I For storing guidelines refer to Medela Breastpump instructions for use or check with your hospital for specific instructions.

Material: Bottle (Polypropylene), Lid (High-density polyethylene)

Disposal: According to local regulations

REF **200.2957** = Multibox of 12 Sets (total 480 bottles/lids)

CLEAN BY MANUFACTURING

Unless bottle is opened or damaged



See bottle

