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EN

Instructions for use Symphony and Lactina One-Day Pump Set

To express breastmilk with the Medela Symphony or Lactina One-Day Pump Set.

| Intended use |
|---|
| To express breastmilk with the Medela Symphony or Lactina One-Day Pump Set. |
| Important Safeguards |
| Read all instructions before use |
| • This is a single user product. Use by more than one person may present a health risk as the components cannot be sterilised for safe re-use between multiple users. |
| • Follow the cleaning instructions unless you are told otherwise by your healthcare professional. |
| • Do not use when package is opened or damaged prior to first use. |
| • Use Medela original accessories only. |
| • Never leave accessories unattended and keep out of reach of children. |
| • If problems or pain occur, consult your lactation consultant or physician. |
| • For further information refer to Symphony or Lactina instructions for use. |

• Choose the right breastshield size for maximum comfort and efficiency while pumping. For breastshield fitting guidelines refer to Symphony or Lactina instructions for use.

- Use a disposable breastmilk bottle according to the individual needs (various bottle sizes available).
- Additionally for use with Symphony: Symphony Membrane Cap and Protective Membrane. For use with Lactina: Lactina Piston Cylinder incl. Cylinder Holder. We recommend to exchange/clean these extra parts regularly or in case of an overflow, immediately.

- Assemble all parts, connect with breastpump and start pumping.
- A new Symphony and Lactina One-Day Pump Set is required:
 - for each mother
 - after a maximum of 8 pumping sessions within 24 hours
 - every 24 hours
 - if milk accidentally gets into the tubing
 - if parts are missing or damaged
- A new disposable breastmilk bottle is required:
 - for each pumping session

| Assembly/Disassembly instructions |
|---|
| <p>Important</p> <ul style="list-style-type: none">• Use a new Symphony and Lactina One-Day Pump Set is required: <ul style="list-style-type: none">– for each mother – after a maximum of 8 pumping sessions within 24 hours – every 24 hours – if milk accidentally gets into the tubing – if parts are missing or damaged • A new disposable breastmilk bottle is required: <ul style="list-style-type: none">– for each pumping session |

| Assembly/Disassembly instructions |
|---|
| <p>Important</p> <ul style="list-style-type: none">• Use a new Symphony and Lactina One-Day Pump Set is required: <ul style="list-style-type: none">– for each mother – after a maximum of 8 pumping sessions within 24 hours – every 24 hours – if milk accidentally gets into the tubing – if parts are missing or damaged • A new disposable breastmilk bottle is required: <ul style="list-style-type: none">– for each pumping session |

Cleaning and Hygiene

Important

- Use drinking water quality for cleaning.
- Clean all parts that come in contact with the breast and tubing immediately after each use.
- Wash parts with soap and water. Rinse parts up of milk residues and to prevent growth of bacteria.

Caution

- The Symphony and Lactina One-Day Pump Set is not autoclavable.
- Use of other sanitising methods (e.g. cold water disinfection) is not recommended and may damage some parts.

Prior to first use

- The Symphony and Lactina One-Day Pump Set does not need cleaning prior to first use as it is provided Ready-to-Use.

After each use

- Disassemble all parts prior to cleaning.
- Remove white valve membrane from breastshield connector.
- Rinse breastshield, connector and white valve membrane in clear water.
- Handwash parts thoroughly and properly with warm, soapy water and rinse in clear water.
- Dry parts with a clean towel.
- The tubing does not need cleaning.
- After cleaning, when parts are not in use, put in a clean zip lock bag, store in a container with lid or wrap in a clean towel.

| Storing/Thawing Breastmilk |
|---|
| <ul style="list-style-type: none">• Label bottle with name, date/time of pumping and amount of breastmilk. • For storing/thawing guidelines refer to Symphony or Lactina instructions for use or check with your hospital for specific instructions. |

Material: Breastshield and Connector (Polypropylene), Valve Membrane (Thermoplastic elastomer), Tubing (Polyvinyl chloride).

Disposal: According to local regulations.

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Important

- Ein neues Symphony and Lactina One-Day Pump Set is required:
 - for each mother
 - after a maximum of 8 pumping sessions within 24 hours
 - every 24 hours
 - if milk accidentally gets into the tubing
 - if parts are missing or damaged
- A new disposable breastmilk bottle is required:
 - for each pumping session

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