Instructions for use

**Breastmilk Bottle
with Teat**

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Mode d'emploi

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Breastmilk Bottle with Teat

Ideal for collecting, storing and feeding breastmilk

Make sure breastfeeding is well established before introducing a teat.

Cleaning and Hygiene

- Use drinking water quality for any of the following described cleaning procedures.
- Clean parts immediately after use to avoid dry up of milk residues and to prevent from growth of bacteria.
- Take care not to damage parts when using a brush.
- Inspect all items before each use.
- Replace parts if damaged or worn out.
- For hygiene reasons, we recommend replacing teats after 3 months.
- Note: Food colorings may discolor components which will not affect performance.
- Follow these cleaning instructions unless you are told otherwise by your healthcare professional.

Prior to first use and once daily

- Disassemble, wash and rinse all parts.
- Place parts in a saucepan of sufficient capacity.
- Fill with cold water until the parts are well covered. Optionally, a tea spoon of citric acid may be added to avoid a build-up of limescale.
- Heat it up and allow to boil for 5 minutes.
- Remove from stove and drain water off.
- Dry parts with a clean towel.

As an alternative to boiling:

Dishwasher: Clean parts in top rack.

Microwave: Use Medela Quick Clean™ Microwave Bags or other microwave-safe device.

Steam sterilizer: Follow the manufacturer's instructions.

After each use

- Disassemble and wash all parts in warm, soapy water.
- Rinse in clear water. Dry parts with a clean towel.
- After cleaning, when parts are not in use, put in a clean zip lock bag or store in a container with lid.
- Alternatively, the parts can be wrapped in a clean paper or cloth towel.
- Do not place in direct sunlight.
- Avoid touching the inside of bottles and lids.

Storing / thawing breastmilk

(For Healthy Term Babies)

Freshly expressed breastmilk

Room Temperature	Do not leave at room temperature
Refrigerator	3 – 5 days at approx. 4 °C
Refrigerator Freezer Compartment	6 months at approx. –16 °C
Deep Freezer	12 months at approx. –18 °C

Thawed breastmilk (previously frozen)

Room Temperature	Do not store
Refrigerator	10 hours
Refrigerator Freezer Compartment	Never refreeze
Deep Freezer	Never refreeze

- When freezing do not fill bottles more than $\frac{3}{4}$ full to allow space for expansion.
- Label bottles with the date of pumping.
- Do not store breastmilk in the door of the refrigerator.
- In order to preserve the components of the breastmilk, thaw in the refrigerator overnight or by holding the bottle under running warm water (max. 37 °C).
- Gently shake the bottle to blend any fat that has separated.
- Do not thaw frozen breastmilk in a microwave or in a pan of boiling water (danger of burning).

For your child's safety and health

Always use this product under adult supervision. All parts which are not in use must be kept out of children's reach. Never use a teat as a soother. Continuous and prolonged sucking of fluids can cause tooth decay. Always check the temperature of food before feeding it to your baby (risk of burning). Do not warm milk in a microwave oven as this may cause uneven heating and could scald your baby.

Material Bottle: Polypropylene
 Teat: Silicone

Disposal According to local regulations

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