Instructions for use

**Reusable Hospital Teat**

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Mode d'emploi

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## Ideal for feeding breastmilk

### Cleaning and Hygiene

- Use drinking water quality for any of the following described cleaning procedures.
- Clean parts immediately after use to avoid dry up of milk residues and to prevent from growth of bacteria.
- Take care not to damage parts when using a brush.
- Inspect all items before each use.
- Replace parts if damaged or worn out.
- For hygiene reasons, we recommend replacing teats after 3 months.
- Note: Food colorings may discolor components which will not affect performance.

### Cleaning

These cleaning instructions are general recommendations and may individually be adjusted based on the hospital's specific directives and cleaning practices.

### Important

It is recommended to provide each baby with an own teat. The teat must be sterilized before being used for the first time. It must be cleaned, disinfected and sterilized (e.g. autoclave) after each use or, latest each time it is handed over to the next baby. Use by more than one baby without adequate sterilization may pose a health risk (e.g. risk of cross-contamination) and voids the warranty.

### Cleaning

Thoroughly wash teats in warm, soapy water. Rinse in clear water.

### Disinfecting

Disinfect by washing in a thermic washing machine (at approx. 93°C).

### Sterilizing

Prior sterilizing visually check teats if clean and intact. Sterilize all teats by autoclaving at 134°C for 3 (or 18) minutes or at 125°C for 15 – 20 minutes. Do not stack teats or apply external pressure to avoid damages.

### **For your child's safety and health**

Always use this product under adult supervision. All parts which are not in use must be kept out of children's reach. Never use a teat as a soother. Continuous and prolonged sucking of fluids can cause tooth decay. Always check the temperature of food before feeding it to your baby (risk of burning). Do not warm milk in a microwave oven as this may cause uneven heating and could scald your baby.

Material    Silicone

Disposal    According to local regulations

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