## Baby Cup

Indications:
The Baby Cup Feeder is a short term use product for feeding breastmilk to babies requiring medication or small supplements. It is also suitable for babies that cannot feed at the breast directly, e.g. weak babies, premature babies or babies with neurological disorders.
Cleaning and Hygiene

##### Important

- Use drinking water quality for any of the following described cleaning procedures.
- Clean cup immediately after use to avoid dry up of milk residues and to prevent from growth of bacteria.
- Replace cup if damaged or worn out.

Cleaning
These cleaning instructions are general recommendations and may individually be adjusted based on the hospital's specific directives and cleaning practices.

Important
It is recommended to provide each baby with an own cup. The cup must be sterilized before being used for the first time. It must be cleaned, disinfected and sterilized (e.g. autoclave) after each use or, latest each time it is handed over to the next baby. Use by more than one baby without adequate sterilization may pose a health risk (e.g. risk of cross-contamination) and voids the warranty.

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Bruksanvisning
<b>Drikkebæger</b>

For your child's safety and health
Before first use, please seek consultation from your healthcare professional. Always check the temperature of milk before feeding it to your baby (risk of burning). Do not warm milk in a microwave oven as this may cause uneven heating and could scald your baby.

Material: Polypropylene
Disposal: According to local regulations
CE-mark according to 93/42/EEC

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