

- After each use**
- Always unplug the appliance from the mains before cleaning it.
 - Do not clean the appliance when hot.
 - Wait 15 minutes after use to allow the Steam Sterilizer to cool down.
 - Ensure that the plug does not come into contact with water.
 - Wipe the inside and outside of the Steam Sterilizer with a damp (not wet) cloth.

*Note: Use water of drinking quality for any described cleaning procedures
Do not clean with abrasive cleaners
Never place the Steam Sterilizer directly under tap water
Never clean the Steam Sterilizer in a dishwasher*

Cleaning After a while in use lime scale will build-up around the hotplate of the Steam Sterilizer. This should be removed at regular intervals.

Every month (De-scale)

Version 1

- Mix **100 ml/3.3 oz** of water with citric acid* according to manufacturer's instructions and pour solution into empty Steam Sterilizer.
- DO NOT place bottles or accessories into the Steam Sterilizer when de-scaling.
- DO NOT put lid (2) on.
- Press On-/Off-Button (6). The Cleaning process has started.
- After 1-2 minutes unplug and leave the solution to cool for 30 minutes before emptying the Steam Sterilizer and rinsing with tap water.
- After this run a full sterilizing cycle, without content, with the lid (2) on and using exactly 100 ml of water.
- Wipe the inside and outside of the Steam Sterilizer with a damp (not wet) cloth only.

Version 2

- Pour 50 ml/1.8 oz of vinegar mixed with 50 ml/1.8 oz of cold water into the Steam Sterilizer.
- DO NOT place bottles or accessories into the Steam Sterilizer when de-scaling
- DO NOT start the sterilizing cycle.
- Leave liquid in the unit until lime scale has dissolved.
- Empty the unit and rinse thoroughly with clear water.
- Wipe the inside and outside of the Steam Sterilizer with a damp (not wet) cloth only.

**Citric acid is available from most drugstores and pharmacies*

Note: Do not use anything other than citric acid or vinegar as this may cause irreparable damage