Instructions for use

Preemie⁺ Symphony Initiation Card

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Preemie⁺ Symphony Initiation Card

Indication for use

Assists breastpump dependent mothers of premature infants in initiating and maintaining their milk supply over time

Assembly Instructions

- Remove Card and Cord protector on the back of the pump
- Exchange the program card 2.0 with the Preemie⁺ card
- Mark the pump with the Preemie⁺ sticker
- Put card and cord protector back in place

Operation

For Preemie⁺

Press On/Off, followed by the let down button (within 10 sec.)

For Standard

Press On/Off

Preemie⁺ 1.0

- Use from the first time postpartum until you have reached initiation ❶
- Afterwards, the Standard program needs to be used
- Use the Pumping Log to control and track each pumping session

The program runs continuously for 15 minutes and is a sequence of different stimulation, expression and pause phases. The display shows „Done“ if the program is finished.

Standard 2.0

- Use after you have reached Initiation ❶
- Use the Pumping Log to control and track each pumping session

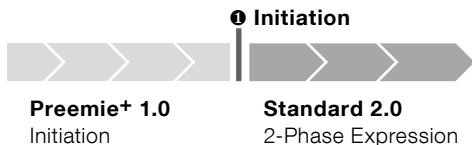
The Standard program operates 2-Phase Expression starting with a stimulation phase, followed by expression. For detailed description see the Symphony instruction manual.

Vacuum Level

Adjust the vacuum during pumping to your comfort level by turning the one-knob control clockwise (increase) or counter-clockwise (decrease).

Pumping

Select one of the available pumping programs

❶ You have reached **initiation** if you have pumped 20ml in each of three consecutive pumping sessions.

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