*No. 1 choice  
of hospitals  
& mothers*

# Pump In Style® Advanced

- EN** Instructions for use
- DE** Gebrauchsanweisung
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- NL** Gebruiksaanwijzing
- ES** Manual de instrucciones


- EN** Parts List XIV
- DE** Teilleiste XIV
- FR** Liste des pièces XIV
- IT** Elenco componenti XIV
- NL** Onderdelenlijst XIV
- ES** Lista de piezas XIV

<b>EN</b>	Instructions for use	2
<b>DE</b>	Gebrauchsanweisung	11
<b>FR</b>	Mode d'emploi	20
<b>IT</b>	Istruzioni per l'uso	29
<b>NL</b>	Gebruiksaanwijzing	38
<b>ES</b>	Manual de instrucciones	47
	EMC	56

## I Pump In Style® Advanced Breastpump

Mother's milk is the best nutrition for babies. Pump In Style Advanced gives you and your baby optimal support during the breastfeeding period. The attractive breastpump in its backpack is especially suitable for the active, working mother who wants to continue breastfeeding her baby for as long as possible.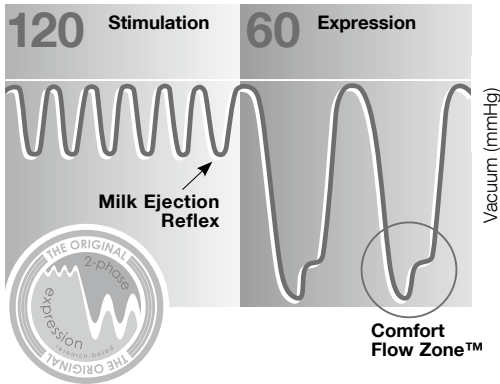

**Pump In Style Advanced** has **2-Phase Expression®**, the only research based breastpump technology to mimic a baby's natural nursing rhythm.

**Stimulation:** Fast pumping rhythm to start your milk flowing.

**Expression:** Slower pumping rhythm to express your milk gently and efficiently.

**Comfort Flow Zone™:** For maximum comfort and efficiency – gentle to the breast tissue.

**Proven to feel most like your baby and to reduce your pumping time.**

## II Uses

- Collect breastmilk at work or during other absences from the baby.
- Maintain a milk supply if breastfeeding is temporarily interrupted.
- Relieve engorgement.
- For home use and on the go.

## III Important Safeguards

*When using electrical devices, basic safety precautions should always be followed, including the following:*

*Read all instructions before use.*

- Always unplug electrical devices immediately after using.
- Separation from the mains is only assured through the disconnection of the mains adapter from the wall socket outlet.
- Do not use while bathing or showering.
- Do not place in or drop motor unit into water or other liquid.
- Do not reach for any electrical device if it has fallen into water. Unplug immediately.
- Do not get the product wet.

- Never operate an electrical device if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.
- Keep the mains adapter away from heated surfaces.
- Make sure the voltage of the mains adapter is compatible with the power source.
- Only use the mains adapter that comes with the Pump In Style Advanced Breastpump.
- Plug the mains adapter into Pump In Style Breastpump first and then into wall socket.
- The product should never be left unattended when plugged into a power source.
- Check pump kit components for wear or damage.
- **Use Medela original accessories only.**
- Use the product only for its intended use as described in this instruction manual.
- Supervision is necessary when this product is used in the vicinity of children.
- If problems or pain occur, consult your lactation consultant or physician.
- Equipment not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide.
- Portable and mobile radio frequency communications equipment can affect the Pump In Style Advanced Breastpump.
- The Pump In Style Advanced Breastpump needs special precautions regarding EMC (Electromagnetic compatibility) and needs to be installed and put into service according to the EMC information provided in these instructions for use.

SAVE THIS INSTRUCTION MANUAL

## IV Assembly Instructions

Make sure breastshield and connector are securely attached.


Snap membranes onto valves until membranes lie completely flat.

Push assembled valve and membrane onto breastshield. Ensure sidewise position of valve as indicated on picture.


## IV Assembly Instructions

4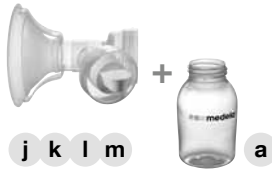

Screw bottles into breastshields.

5

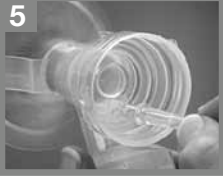Insert hard adapter ends of tubing into openings in breastshields.

6

Insert other ends of tubing into ports on diaphragm cap. Remove tubing by grasping the ends and twisting until tapered ends loosen.

*NOTE: Do not remove tubing by forcefully pulling the tube or pulling at an angle.*

7

- If single pumping, insert port plug into the unused port.
- If double pumping, push port plug onto post.

### Electrical operation

8a

Plug the electrical connector on the end of mains adapter or Battery Pak cord into power jack on breastpump. Plug the mains adapter into a standard electrical outlet.

### Battery operation

8b

Slide off battery covers. Insert the batteries in the correct position as indicated inside the Battery Pak. Push the battery in and down in one motion until the battery snaps into place. Five batteries must be inserted on each side of the Battery Pak. Replace battery covers. The Battery Pak was designed for safe use while stored in its pouch. The pouch fits inside the main bag or motor bag, along with the mains adapter. Or for easier access, the velcro strap allows for attachment to the pump shoulder strap.

*The Battery Pak uses 10 AA alkaline or rechargeable batteries. Each set of batteries will provide approximately 2 hours of pumping time. It is recommended that batteries be replaced or recharged between the 10th and 12th pumping session.*

## V Pumping

Be comfortably seated and relax. Use a footrest, when pumping or when feeding your baby. Elevating your feet puts you into proper sitting position to avoid stress on the lower back and perineum.

Center breastshields over your nipples.


- Do not tilt or overfill bottles when pumping.
- Take care not to kink tubing(s) while pumping.

Switch on the breastpump by turning the switch clockwise.

The pump will begin pumping in the **Stimulation Phase**.

Adjust the vacuum during pumping to your comfort level by turning the switch clockwise (increase) or counterclockwise (decrease).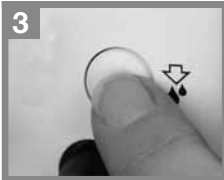

Once your milk starts to flow or you sense milk ejection, press the let-down button to switch to the **Expression Phase**. If you do not press the let-down button, the pump will automatically switch to the Expression Phase after two minutes.

Set your **Maximum Comfort Vacuum™** which is the highest vacuum you can use and still feel comfortable (this is different for every mother). To find your Maximum Comfort Vacuum increase vacuum until pumping feels slightly uncomfortable (not painful), then decrease slightly.

If you need to return to the stimulation phase for any reason, simply press the let-down button. You may return back to the expression phase quickly by pressing the let-down button again.

**Research has shown that you will pump more efficiently – meaning get more milk in less time – when pumping at Maximum Comfort Vacuum™ in the Expression Phase.**

**Tip:** Some moms prefer going back and forth between stimulation and expression to achieve second let-downs throughout a pumping session. You can do this by pressing the let-down button at any time.

Once finished pumping, switch off the breastpump, unplug it, and disassemble the pump kit items. Close the bottle and store it for later use.

*Use bottle stand to prevent tip over.*

## VI Breastshield Fitting

### Choosing the right size of Breastshield

A correctly fitted breastshield avoids compression of any milk ducts during pumping to ensure optimal breast emptying and maximum milk output.

Your breastshield is the right size if your answer to the following questions is «Yes»:

- Is your nipple moving freely in the tunnel?
- Is minimal or no areolar tissue being pulled into the tunnel of the breastshield?
- Do you see a gentle, rhythmical motion in the breast with each cycle of the pump?
- Do you feel the breast emptying all over?
- Is your nipple pain-free?

If your answer to any of the above questions is «No», try a larger (or smaller) breastshield.

*Medela PersonalFit™ Breastshield: Available in five sizes – because one size does not fit all.*

## VII Manual Pumping

### Follow steps 1 – 4 on page 3/4

5

Slip rubber seal onto piston until it snaps into place – the larger side faces the handle. To ensure proper seal, hold rubber seal firmly and twist piston.

6

Slide piston into cylinder.

7

Screw cylinder into breastshield.

8

Set the vacuum regulator to minimum and start pumping. Adjust the vacuum to your **Maximum Comfort Vacuum™**. This is the highest vacuum you can use and still feel comfortable (different for every mother). To find your Maximum Comfort Vacuum increase vacuum until pumping feels slightly uncomfortable (not painful), then decrease slightly.

## VIII Removing and Securing the Diaphragm Cap

Only remove diaphragm cap when cleaning is necessary.

### To remove the diaphragm cap

- Turn off the pump and unplug from power source.
- Remove tubing (see instructions on page 4).
- Snap off the diaphragm cap using the thumb tab.

### To secure the diaphragm cap

Hold back of pump with one hand while pushing on the diaphragm cap until it snaps into place.

*NOTE: The diaphragm cap must be completely snapped together at all connection points to ensure adequate vacuum and optimum pump performance.*

## IX Cleaning and Hygiene

Follow these cleaning instructions unless you are told otherwise by your healthcare professional.

### Important

Disassemble all parts prior to cleaning and sanitizing. Especially be sure to remove the yellow valve head from the breastshield and separate the white valve membrane from the valve head.

### Prior to first use and once daily

- Disassemble, wash and rinse all parts of your pump kit that come in contact with the breast and milk (except motor unit, bag, etc.).
- Place parts in a saucepan of sufficient capacity.
- Fill with cold water until the parts are well covered.
- Heat it up and allow to boil for 5 minutes.
- Remove from stove, drain water off and allow parts to cool and air-dry on a clean towel.

Alternatively, the disassembled parts can be sanitized in top rack of dishwasher.

### After each use

- Disassemble and wash all parts of your pump kit that come in contact with the breast and milk in warm, soapy water (except motor unit, bag, etc.).
- Rinse in clear water and allow parts to air-dry on a clean towel.
- Use a damp (not wet) cloth to wipe the motor unit and bag of breastpump after each use.

**Tubing care** Inspect the tubing after each pumping session for condensation and/or milk. If condensation appears in the tubing  
Let the pump run after you have completed pumping with tubings attached for 1–2 minutes or until dry.  
If milk gets into the tubing  
If milk accidentally gets into the tubing turn off the pump and unplug from power source. Remove and clean the tubing according to the instructions «Prior to first use and once daily». Shake water droplets out and hang to air-dry. For faster drying, attach the tubing to pump and run for 1–2 minutes or until dry.

**Hints** Wrap parts with a clean towel or store in a container when not in use.

- Hygiene**
- Wash hands thoroughly with soap and water before touching the breast and pump parts.
  - Avoid touching the inside of bottles and lids.
  - Clean the breast and surrounding area with a damp cloth. Do not use soap or alcohol which can dry the skin and cause cracked nipples.

## X Storing / Thawing Breastmilk

Check with your hospital for specific instructions.

### Breastmilk Storage (For Healthy Term Babies)

	Room Temperature	Refrigerator	Refrigerator Freezer Compartment	Deep Freezer
	<i>Do not leave at room temperature</i>	<i>3 – 5 days at approx. 4° C</i>	<i>6 months at approx. -16° C</i>	<i>12 months at approx. -18° C</i>
Thawed breastmilk (previously frozen)	<i>Do not store</i>	<i>10 hours</i>	<i>Never refreeze</i>	

- When freezing do not fill bottles/bags more than 3/4 full to allow space for expansion.
- Label bottles/bags with the date of pumping.
- Do not store breastmilk in the door of the refrigerator.
- In order to preserve the components of the breastmilk, thaw in the refrigerator overnight or by holding the bottle/bag under running warm water (max. 37° C).
- Gently shake the bottle/bag to blend any fat that has separated.
- Do not thaw frozen breastmilk in a microwave oven or in a pan of boiling water (danger of burning).

### In the Pump In Style Advanced Cooler Carrier

One contoured cooling element is necessary for the Pump In Style Advanced Cooler Carrier. As soon as breastpumping is finished, place the milk bottle or bag in the compartment with the cooling element. Milk will keep cool there until you return home later that day to transfer the bottles/bags to the refrigerator or freezer.

## XI Troubleshooting

### If there is low or no suction

#### Make sure

- the breastshield forms a complete seal around the breast.
- all connections of the kit and breastpump are secure.
- the white valve membrane is laying flat against the yellow valve head. Check that the valve head is clean and not damaged. See «Assembly Instructions».
- the diaphragm cap or faceplate on the front of the breastpump fits evenly and securely.
- the ends of the tubings are fit securely into the backs of the breastshields and into the ports on the diaphragm cap or faceplate.
- If single pumping, make sure the port plug is placed into one of the single pumping ports.

*Take care not to kink tubing. If milk is seen backing up into tubing, check that tubing is not kinked or pinched.*

### If there is an overflow

#### If milk accidentally gets into the tubings or behind the diaphragm cap

1. Turn off the pump and unplug from power source.
2. Refer to cleaning instructions «Tubing care».
3. Use a damp (not wet) cloth to wipe the diaphragm and bag. Do not immerse the motor unit or any of the bags in water. If necessary clean the pump kit.
4. When components are completely dry, reassemble breastpump and kit.

## XII Warranty

This is a single user product. Use by more than one person may present a health risk and voids the warranty. 1-year warranty for the unit from the date of purchase, excluding pump kit, battery pak, backpack and cooler carrier. The manufacturer is not liable for any damage or consequential damage caused by incorrect operation, inappropriate usage or use by unauthorized persons.

## XIII Disposal

The unit is made of various metal and plastics. Before disposal, the device is to be rendered unusable and it must not be disposed of as unsorted municipal waste in accordance with local regulations. Use your local return and collection system for waste electrical and electronic equipment (incl. batteries). Improper disposal may have harmful effects in the environment and on public health.

## Article numbers

a	Bottle (150 ml)	811.0021
b	Cooling Element	811.7010
c1	Faceplate Assembly (EURO, UK, AUS/NZ)	600.7095
c2	Faceplate Assembly (US/JP)	600.7094
d1	Lid	810.7062
d2	Disc	810.0473
e1	Battery Pak (EURO, UK, AUS/NZ)	901.7006
e2	Battery Pak (US/JP)	901.7004
f	Base Transformer	200.3615
f1	Plug UK 240V	200.3616
f2	Plug Euro 230V	200.3617
f3	Plug US 120V	200.4193
g	Lactina Piston/Cylinder Assembly	920.7041
h	Tubing	800.0652
i	Bottle stand	800.7212
j	SoftFit™ Breastshield (24 mm)	810.0462
k	Connector	810.0402
l	Valve head	810.7088
m	Valve membrane	810.0475

### **Accessories** (sold separately)

Car adapter

928.7001

*Does not contain Bisphenol-A, DEHP (Phthalate) or Latex.*

*Subject to change.*

*Content may vary from picture.*

## I Pump In Style® Advanced Brustpumpe

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Electromagnetic Compatibility (EMC, IEC 60601-1-2:2001)

IEC 60601-1-2, Table 201

<b>Electromagnetic Emissions</b>		
The Pump In Style® Advanced (PNSA) is intended for use in the electromagnetic environment specified below. The customer or the user of the PNSA should assure that it is used in such environment.		
<b>Emission Tests</b>	<b>Compliance</b>	<b>Electromagnetic environment - guidance</b>
RF emissions CISPR 11	Group 1	The PNSA uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.  The PNSA is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations / flicker emissions IEC 60000-3-3	Complies	

**Warning:** The PNSA should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the PNSA should be observed to verify normal operation in the configuration in which it will be used. The PNSA was not tested for immunity to electromagnetic disturbances.

- EN** Technical specifications
- DE** Technische Daten
- FR** Caractéristiques techniques
- IT** Caratteristiche tecniche
- NL** Technische gegevens
- ES** Especificaciones técnicas

	vacuum (approx.)	
	-50 ...	-250 mmHg
	-7 ...	-33 kPa
	54 ...	120 cpm

	In	Out
	Euro 230 V	12V $\overline{\overline{\overline{\quad}}}$
	UK 240 V	1A
	50 Hz ~ AUS	
	NZ	
60 Hz ~ US	120 V	~ alternating current
	JP	$\overline{\overline{\overline{\quad}}}$ direct current

	Car adapter Autoadapterkabel Câble de branchement sur allume-cigares Adattatore per automobile Auto-adapter Adaptador automóvil	12 V DC
--	--	---------

	395 x 280 x 165 mm
--	--------------------

	kPa
--	-----

	Operation Betrieb En Service Utilizzo Gebruik Funcionamiento
--	---

	Transport / Storage Transport / Lagerung Transport / Stockage Transport / Stoccaggio Transport / Opslag Transporte / Almacenamiento
--	--

	Operation Betrieb En Service Utilizzo Gebruik Funcionamiento
---	---

	Transport / Storage Transport / Lagerung Transport / Stockage Transport / Stoccaggio Transport / Opslag Transporte / Almacenamiento
--	--

	4.1 kg
--	--------

	10 x 1.5 V Alkaline (AA, LR6) or 10 x 1.2 V Ni Mh/ Ni Cd (Rechargeable AA, LR6)
--	---

	Protection class II Type B	
--	-------------------------------	---

	FDA (GMP) ISO 9001 / ISO 13485 CD (93/42/EEC)	
--	---	---

(except 800.0717)

Please see accompanying papers  
 Begleitpapiere beachten  
 Respecter les consignes des documents d'accompagnement  
 Consultare la documentazione allegata  
 Zie bijgevoegde documenten  
 Consultar documentación adjunta

Must not be disposed of together with household refuse  
 Darf nicht zusammen mit Hausmüll entsorgt werden  
 Ne doit pas être éliminé avec les ordures ménagères  
 Non deve essere smaltito con i rifiuti domestici  
 Mag niet samen met huisvuil weggegooid worden  
 No eliminar junto con la basura doméstica

[www.medela.com](http://www.medela.com)

## INTERNATIONAL SALES

Medela AG, Medical Technology  
Lättichstrasse 4b  
6341 Baar  
Switzerland  
Phone +41 (0)41 769 51 51  
Fax +41 (0)41 769 51 00  
[ism@medela.ch](mailto:ism@medela.ch)  
[www.medela.com](http://www.medela.com)

## AUSTRALIA

Medela Pty Ltd, Medical Technology  
3 Arco Lane, Heatherton  
Vic 3202 / Australia  
Phone +61 (0) 3 9552 8600  
Fax +61 (0) 3 9552 8699  
[contact@medela.com.au](mailto:contact@medela.com.au)  
[www.medela.com.au](http://www.medela.com.au)

## FRANCE

Medela France Sari  
14, rue de la Butte Cordière  
91154 Etampes cedex  
France  
Phone +33 (0)1 69 16 10 30  
Fax +33 (0)1 69 16 10 32  
[info@medela.fr](mailto:info@medela.fr)  
[www.medela.fr](http://www.medela.fr)

## GERMANY & AUSTRIA

Medela Medizintechnik  
GmbH & Co. Handels KG  
Postfach 1148  
85378 Eching  
Germany  
Phone +49 (0)89 31 97 59-0  
Fax +49 (0)89 31 97 59 99  
[info@medela.de](mailto:info@medela.de)  
[www.medela.de](http://www.medela.de)

## ITALY

Medela Italia Srl  
Via Turini, 19 - Loc. Bargellino  
40012 Calderara di Reno (BO)  
Italy  
Phone +39 051 72 76 88  
Fax +39 051 72 76 89  
[info@medela.it](mailto:info@medela.it)  
[www.medela.it](http://www.medela.it)

## NETHERLANDS & BELGIUM

Medela Benelux BV  
Uilenwaard 31  
5236 WB Empel  
Netherlands  
Phone +31 73 690 40 40  
Fax +31 73 690 40 44  
[info@medela.nl](mailto:info@medela.nl), [info@medela.be](mailto:info@medela.be)  
[www.medela.nl](http://www.medela.nl), [www.medela.be](http://www.medela.be)

## SPAIN & PORTUGAL

Productos Medicinales Medela, S.L.  
c/ Manuel Fernández Márquez, 49  
08918 Badalona (Barcelona) / Spain  
Phone +34 93 320 59 69  
Fax +34 93 320 55 31  
[info@medela.es](mailto:info@medela.es), [info@medela.pt](mailto:info@medela.pt)  
[www.medela.es](http://www.medela.es), [www.medela.pt](http://www.medela.pt)

## SWITZERLAND

Medela AG, Medical Technology  
Lättichstrasse 4b  
6341 Baar  
Switzerland  
Phone +41 (0)848 633 352  
Fax +41 (0)41 769 51 00  
[contact@medela.ch](mailto:contact@medela.ch)  
[www.medela.ch](http://www.medela.ch)

## UNITED KINGDOM

Medela UK Ltd.  
Huntsman Drive  
Northbank Industrial Park  
Irlam, Manchester M44 5EG  
United Kingdom  
Phone +44 870 950 5994  
Fax +44 870 389 2233  
[info@medela.co.uk](mailto:info@medela.co.uk)  
[www.medela.co.uk](http://www.medela.co.uk)