

- Never operate an electrical device if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.
- Keep the mains adapter away from heated surfaces.
- Make sure the voltage of the mains adapter is compatible with the power source.
- Only use the mains adapter that comes with the Pump In Style Advanced Breastpump.
- Plug the mains adapter into Pump In Style Breastpump first and then into wall socket.
- The product should never be left unattended when plugged into a power source.
- Check pump kit components for wear or damage.
- **Use Medela original accessories only.**
- Use the product only for its intended use as described in this instruction manual.
- Supervision is necessary when this product is used in the vicinity of children.
- If problems or pain occur, consult your lactation consultant or physician.
- Equipment not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide.
- Portable and mobile radio frequency communications equipment can affect the Pump In Style Advanced Breastpump.
- The Pump In Style Advanced Breastpump needs special precautions regarding EMC (Electromagnetic compatibility) and needs to be installed and put into service according to the EMC information provided in these instructions for use.

SAVE THIS INSTRUCTION MANUAL

IV Assembly Instructions



Make sure breastshield and connector are securely attached.




Snap membranes onto valves until membranes lie completely flat.



Push assembled valve and membrane onto breastshield. Ensure sidewise position of valve as indicated on picture.

IV Assembly Instructions

4

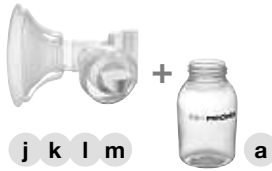

Screw bottles into breastshields.

5



Insert hard adapter ends of tubing into openings in breastshields.

6



Insert other ends of tubing into ports on diaphragm cap. Remove tubing by grasping the ends and twisting until tapered ends loosen.
NOTE: Do not remove tubing by forcefully pulling the tube or pulling at an angle.

7



- If single pumping, insert port plug into the unused port.
- If double pumping, push port plug onto post.

Electrical operation

8a



Plug the electrical connector on the end of mains adapter or Battery Pak cord into power jack on breastpump. Plug the mains adapter into a standard electrical outlet.

Battery operation

8b



Slide off battery covers. Insert the batteries in the correct position as indicated inside the Battery Pak. Push the battery in and down in one motion until the battery snaps into place. Five batteries must be inserted on each side of the Battery Pak. Replace battery covers. The Battery Pak was designed for safe use while stored in its pouch. The pouch fits inside the main bag or motor bag, along with the mains adapter. Or for easier access, the velcro strap allows for attachment to the pump shoulder strap.

The Battery Pak uses 10 AA alkaline or rechargeable batteries. Each set of batteries will provide approximately 2 hours of pumping time. It is recommended that batteries be replaced or recharged between the 10th and 12th pumping session.